

Maintenance

To be performed by ICU Staff

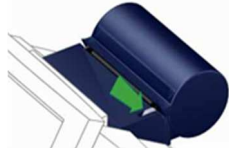
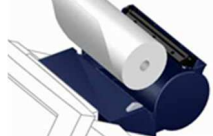
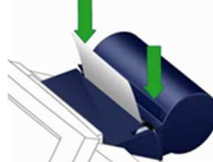
Cleaning

The analyser external surfaces should always be kept clean of blood and any liquids. All users need to ensure that the instrument is cleaned with a wrung out azowipe.

Auto Flush

If a sample is run that is suspected of having a clot. Press the *Auto-Flush* box that appears at the bottom of the main screen.

Change Printer Paper

1. Press the release button to unlock the lid. Open the lid by pressing it backwards and remove any leftover paper from the printer.	
2. Place the new roll in the printer so that the paper unreels from underneath the roll.	
3. Make sure that some of the paper will extend out of the printer when the cover is closed. Then close the cover by pulling the cover forwards and pressing it down until a click is heard.	

To be performed by the laboratory staff

Weekly Maintenance

Various tasks will alert in the *Other Activities* box when they need to be performed. The box will have an orange traffic light when due.

These include:

- Clean inlet
- Clean sample mixer

Follow the instructions and videos on the screen. Ensure they are recorded as completed to allow the traffic light to go green and keep track of completed tasks.

'As required' tasks will be prompted on screen as part of the troubleshooting process and may include:

- Change inlet gasket
- Change inlet probe
- Change connection gasket
- Manual flush

Follow the instructions and videos on the screen to complete.