







Running a sample on the ABL90

<p>1. Make sure the analyser is in READY. Ensure all analytes required are green.</p>	
<p>2. Enter or scan your Staff ID number</p>	
<p>3. Put the syringe in the sample mixer. It automatically mixes with the magnetic ball for 7 seconds. Wait until the light starts blinking. Remove the syringe.</p>	
<p>4. Select syringe. Capillary not available (send to lab) The analyser opens the inlet.</p>	
<p>5. Place safePICO syringe in the centre of the inlet gasket. Push the syringe by its barrel towards the analyser as far as it will go. The needle will go through the lid. Hold it there while the blood is automatically aspirated. Remove the sample when you hear the beep.</p>	
<p>6. Scan the encounter number on the patient's bradma label. Confirm correct patient from popup box. Select Arterial or Venous Enter O2 status as a % and temperature (if required)</p>	
<p>7. The patient result will be displayed. Print if required. Select <i>messages</i> box if one or more analytes has not resulted to find out the reason why. Log Off</p>	